

# CueSee® CO-OX

## Sample Preparation

- 1** Place vial at room temperature for at least 15 minutes but not longer than 60 minutes before testing.




- 2** To activate press the button firmly until it is completely pushed in and level with the indentation.



- 3** Gently tilt the vial to a cap down position and swirl the sample before returning to cap up position and swirl. Repeat 5 times.




- 4** Attach adaptor to end of syringe. Holding the vial in cap down position firmly push the adaptor into the luer opening on the button.

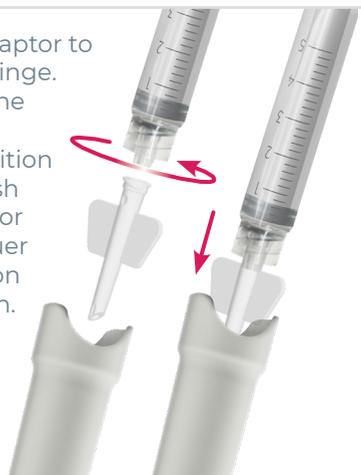

- 5** Turn vial cap up and draw sample into syringe.

Turn vial cap down and remove the syringe leaving the adaptor in the button.

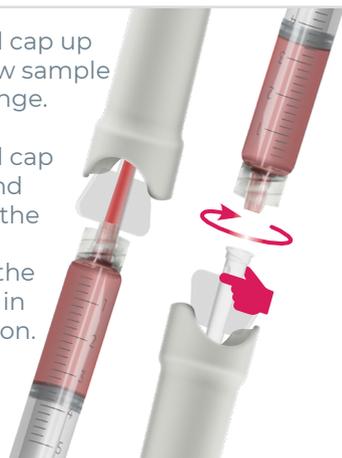

- 6** After preparation the sample is stable for 15 minutes.

