


# Instructions for Urine Culture Samples


1

Allow the agar media, sample vial, and dilution fluid to warm to room temperature.

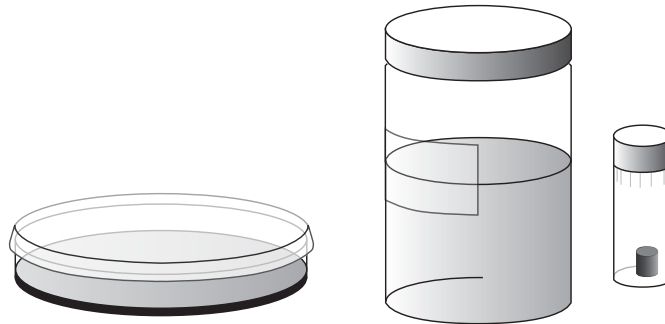


2

For each sample open the lyophilized sample vial. Aseptically transfer the pellet into the corresponding dilution fluid container.

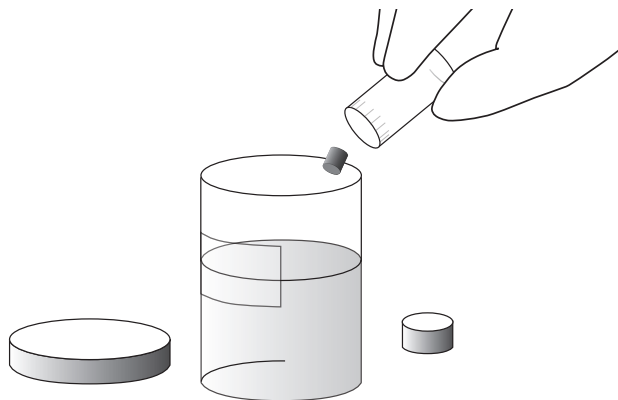

3

Reseal the dilution fluid container, then mix the contents, allow pellet to dissolve for 20 minutes and mix again. No incubation is required before use.

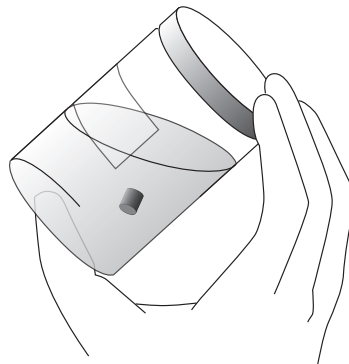

4

The entire contents of the container (which now contains the dissolved lyophilized pellet) simulates a urine sample. Mix and immediately proceed to test as you would a patient sample in your laboratory.